


## Trouble shooting guide for Oxylator FR-300 users.

1. **Connect device to source pressure ( 40psi – 80psi ).**

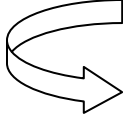


2. **Open source valve to pressurize the FR-300.**  
*You should not hear flow or hissing from the device.  
Do you hear oxygen flow or any hissing from the device?*

**No – go to 3.**

**Yes - make sure the “ORB” is fully disengaged ( counter-clockwise rotated )**

**If problem not corrected, go to 999.**

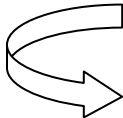

3. **Release “O<sub>2</sub>” button completely, and then attach a test lung ( 1L or 2L ) to Patient Port. Engage “O<sub>2</sub>” button in automatic mode by depressing, then rotating clockwise.**  
*The device should now cycle automatically, and consistently.  
Does the device cycle automatically, and consistently?*

**Yes – the device is functioning properly.**

**No – go to 999.**

999. **Don't use the device.**

**Report the problem, immediately, to your distributor,  
or  
Contact CPR Medical Devices Inc. at (416) 691-2669,  
or  
E-mail HARO@cprmedic.com**

Please have the serial # of the device before contacting for technical support.