

HAZMAT Oxylator Resuscitation Kit

Contents of the Black HAZMAT Oxylator “Kit” Bag:

- 1. Four Oxylator EMXb (“b” indicates butyl rubber components)**
Attach an existing facemask or ET-tube to the Oxylator EMXb, and begin use in manual mode. Adjust the pressure needed to obtain a proper delivered volume of 1-2 seconds of inhalation length before “pop-off”. (Remember that the Oxylator EMXb delivers 500 cc/ml per each one (1) second of inhalation time length) If inhalation length exceeds 2.5 seconds, then use manual mode only.
- 2. Four long hoses, two hoses @ 50 feet, and two hoses @ 30 feet**
These hoses (equipped with female, hand-tightened, DISS connectors) attach to the 50 PSI outputs (with male DISS connections) of your existing regulators mounted on your ‘M’ or ‘H’ tanks of compressed O₂ gas. You may specify a hose length if the 30 feet and 50 feet is not convenient.
- 3. Four shorter hoses of about 7-8 feet in length**
Attach these hoses to each of the four Oxylator EMXb.
- 4. Four sets of Ohio/Ohmeda Male/Female Quick Disconnects**
The ‘green colored’ quick connects are already snapped together, so, separate them by twisting, clockwise, the green band on the Male quick connector, then pull apart. Attach the Female quick connector onto the ends of the long hoses, and attach the Male quick connector onto the shorter 8 foot hoses already attached to the Oxylator EMXb.
- 5. Four SCBA regulators**
If the MMRS depletes their supply of compressed O₂ gas, and a convenient supply of O₂ gas is not available, then these special regulators can be attached to SCBA bottles (provided by a local fire department) to continue the efficient operation of the Oxylator EMXb.
- 6. Four Oxylator Boxes + One Video Tape in a Black Sleeve**
Fill out warranty information for each Oxylator, and then store these Oxylator EMXb boxes (identified by serial #) in a convenient location for possible future use. Please view the videotape (in the black sleeve) for operational characteristics of the Oxylator EMX or Oxylator EM-100. This particular videotape is a recording of an inservice session at the Jacksonville, Florida Fire / EMS who own 157 Oxylators.